


ECG DIAGNOSTIC ELECTRODE


FOR USE WITH:

Product No	Reference No	Product No	Reference No	Product No	Reference No
6500015V	A10005-3-7%	6500581V	A10071-5NB	6500060V	A10040-60
6500017V	A10005-7%NB	6502754V	A10096SG-30	6500061V	A10041-60
6500019V	A10005-60-7%	6502779V	A10097-3	6500062V	A10042-10
6500021V	A10005SG-60	6502781V	A10097-5	6500064V	A10042-60
6500049V	A10031-1-60	6502782V	A10097-5NB	6502683V	Z05MC-3
6500051V	A10031-60	6502785V	A10097-7NB	6502684V	Z05MC-5
6500484V	A10005SG-30	6500070V	A10070-1-60	6502685V	Z05MC-50
6502801V	A10098-10NB	6500071V	A10070-5	6502687V	Z05MT-3
6500636V	A10098-30	6500023V	A10006-3	6502688V	Z05MT-5
6502789V	A10098-5NB	6502776V	A10006-60	6502689V	Z05MT-50
6500057V	A10033-1-60	6500028V	A10006-60T	6502692V	Z05SG-3
6500053V	A10032-1-60	6502843V	P7-3	6502693V	Z05SG-30
6500054V	A10032-5	6500003V	A10000-60	6502694V	Z05SG-5
6500055V	A10032-60	6500004V	A10001	6502695V	Z05SG-50
6500048V	A10026T-60	6502842V	A7-5	6502721V	Z35-3
6502954V	A10005SG-5NB	6502769V	A10040-5	6502722V	Z35-5
6502947V	A10005SG-3NB	6502767V	A10041-5	6502723V	Z35-50
6500072V	A10071-1-60				

INSTRUCTIONS FOR USE

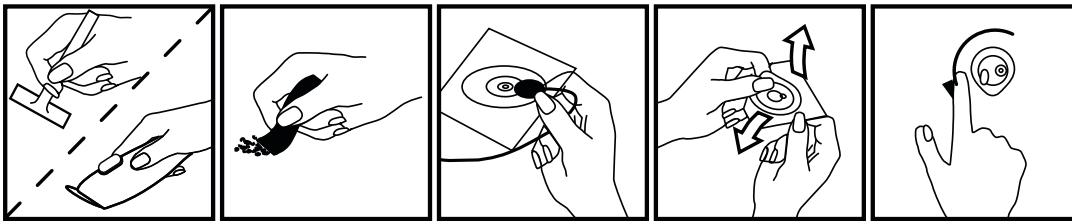

- Shave/Clip
- Prep
- Connect
- Peel
- Apply

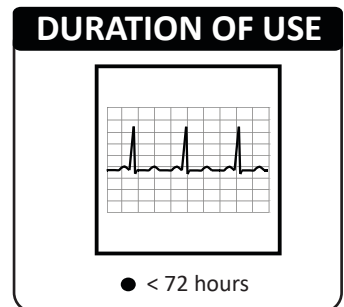

CAUTIONS

- Federal law restricts this device to sale by or on the order of a physician.
- ECG electrodes should only be used by or in consultation with a health care provider familiar with their proper placement and use;
- ECG electrodes may damage the skin if removed carelessly;
- ECG electrodes should be applied only to intact, clean skin (e.g., not over open wounds, lesions, infected, or inflamed areas); and
- ECG electrodes should be properly disposed of after use
- Minor skin irritation can occur with electrode use. If skin irritation occurs, please contact your prescribing physician or health care professional.

PRECAUTIONS

- During surgical procedures, electrodes should be placed as far as possible from any electro-surgical area to minimize unwanted RF current flow.
- Self-adhesive electrodes should be replaced if they no longer stick firmly to the skin.

VML174 (04/2021)



HS.NisshaMedical.com/IFU

