INSTRUCTIONS FOR USE
SINGLE USE ONE-WAY WATER JET CONNECTOR

- | | |
|---|--|
| CAUTION | DO NOT REUSE |
| TEMPERATURE LIMITS | NON-STERILE |
| KEEP AWAY FROM DIRECT SUNLIGHT | MANUFACTURER |
| PROTECT FROM MOISTURE | LATEX FREE
<small>Not made with natural rubber latex</small> |
| LOT CODE | PHT FREE |
| QUANTITY | READ INSTRUCTIONS FOR USE |
| EXPIRY | REFERENCE |
| DO NOT USE IF PACKAGING IS DAMAGED | EC ACCORDING TO DIRECTIVE 93/42/EEC AND UPDATE 2007/47/EC
0197 |
| MEDICAL DEVICE | |

- Do not use tubing that exposes the patient to risk.
- Single-use device that is not designed to be reused.
- Before using this device, please read the instructions for the endoscope you will be using, as well as for any accessories that will be used.

SINGLE USE ONE-WAY WATER JET CONNECTOR


Please read the following information carefully.
Failure to use the product as intended may have harmful consequences for the patient.

Important reminder:
This document describes the instructions for using the “**ENDOtube for Wash Pump**” **Single-Use Patient Connector with Backstop Valve, Compatible with The Canal Water Jet Endoscope**. It contains no references to particular medical procedures. The manufacturer will not be held liable for any problems caused by improper use of the device.

I. INDICATION

The **SINGLE USE ONE-WAY WATER JET CONNECTOR** is designed to connect ENDOtube for Wash Pump tubing and canal water jets. It is intended to replace the connectors delivered with ENDOtube for Wash Pump to replace this component between each patient. It is used during gastrointestinal endoscopy procedures involving an irrigation pump.

- This connector comes in two models:
- 100100 - SINGLE USE ONE-WAY WATER JET CONNECTOR FOR OLYMPUS
 - 100300 - SINGLE USE ONE-WAY WATER JET CONNECTOR FOR FUJIFILM

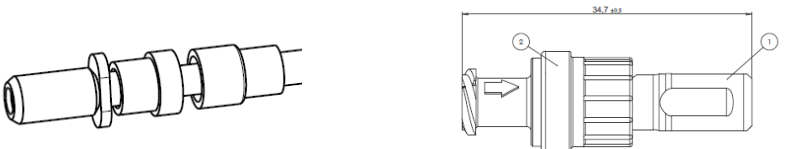ENDOtube for Wash Pump connector

Single-use connector
(to be fitted between each patient)

CONTRAINDICATIONS:
The contraindications are those particular to any endoscopy procedure.

II. STORAGE / PACKAGING

- STORAGE**
- **Room temperature: +10°C to +50°C**
 - **Expiry date: 2 years from production date**

PACKAGING
The connectors are individually packaged in boxes of 50. Once removed from its packaging, it must be used immediately.

III. WARNINGS AND PRECAUTIONS

- The **SINGLE USE ONE-WAY WATER JET CONNECTOR:**
- Must be used as soon as it is opened.
 - Is a single-use device that is not designed to be reused.
 - Must be disposed of as waste in compliance with the country's applicable regulations.
 - Must never be connected or reconnected to an endoscope that has not been washed as per manufacturer instructions.

- To avoid risks of cross-contamination, do not reuse this device.
- Must only be used by qualified endoscopy personnel.

Authorised personnel must use appropriate techniques to avoid contaminating the **SINGLE USE ONE-WAY WATER JET CONNECTOR**. The device comes with a male Luer lock fitting.

Special care must be taken to avoid inadvertently connecting the connector to an existing line in the patient with a female connector.

COMPATIBILITY:

- 100100: OLYMPUS Endoscopes
- 100300: FUJIFILM & PENTAX Endoscopes

IV. USE / MAINTENANCE / CLEANING / DISINFECTION / STERILISATION

USE (must be used with **ENDOfube for Wash Pump** tubing):

It replaces the connector delivered with the ENDOfube for Wash Pump system between patients. (See also the instructions for the endoscope and any accessories used)

It is important that you familiarize yourself with the device and complete a trial run before using it in a clinical context.

1. Open the packaging and remove the product.
2. Screw the connector into the endoscope's canal water jet.
3. Screw the ENDOfube for Wash Pump's Luer lock fitting into the Luer lock on the backstop valve.
4. Power on the irrigation pump and launch the system before any procedure begins.
5. Once the procedure is complete, unplug the ENDOfube for Wash Pump system.
6. Make sure that the device is properly disposed of following use on a single patient and after any cleaning of the endoscope. Dispose of the device according to applicable regulations in your country.

MAINTENANCE:

- Do not use the single-use device if the packaging is damaged.
- No repairs can be carried out on this device.

CLEANING, DISINFECTION, STERILISATION:

This device is for single-use only. Disinfection or cleaning is not necessary.

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