INSTRUCTIONS FOR USE

ENDOTUBE FOR WASH BOTTLE WITH CO2 TUBING

-  **CAUTION**
-  **TEMPERATURE LIMITS**
-  **KEEP AWAY FROM DIRECT SUNLIGHT**
-  **PROTECT FROM MOISTURE**
-  **LOT CODE**
-  **QUANTITY**
-  **EXPIRY**
-  **DO NOT USE IF PACKAGING IS DAMAGED**
-  **MEDICAL DEVICE**
-  **DISPOSE OF AFTER 24 HOURS OF USE**
-  **NON-STERILE**
-  **MANUFACTURER**
-  **LATEX FREE**
Not made with natural rubber latex
-  **PHT FREE**
-  **READ INSTRUCTIONS FOR USE**
-  **REFERENCE**
-  **EC ACCORDING TO DIRECTIVE 93/42/EEC AND UPDATE 2007/47/EC**
0197

-  • Using the tubing for more than 24 hours can cause bacteria to develop in the stagnant water within tubing
- Before using this device, please read the instructions for the endoscope you will be using, as well as for any accessories that will be used.

ENDOTUBE FOR WASH BOTTLE WITH CO2 TUBING

Please read the following information carefully.

Failure to use the product as intended may have harmful consequences for the patient.

Important reminder:


This document contains instructions for using the “**ENDOtube for Wash Bottle CO2 Insufflation,**” tubing for endoscopes. It contains no references to particular medical procedures. The manufacturer will not be held liable for any problems caused by improper use of the device.

I. INTENDED PURPOSE / COMPATIBILITY

ENDOtube for Wash Bottle CO2 Insufflation is designed to supply an endoscope with air or carbon dioxide (CO2) and water during gastrointestinal endoscopy procedures and contains a tube with a filter designed to deliver carbon dioxide (CO2) to an endoscope via a CO2 insufflator. **ENDOtube for Wash Bottle CO2 Insufflation** must be used within 24 hours of opening.

There are eight models of the **ENDOtube for Wash Bottle CO2 Insufflation** available:

- **FLOCO2LUER** is compatible with Olympus endoscopes and insufflators fitted with a female Luer lock fitting.
- **FLPCO2LUER** is compatible with Pentax endoscopes and insufflators fitted with a female Luer lock fitting.
- **FLFCO2LUER** is compatible with FUJIFILM 600 and earlier endoscopes and insufflators fitted with a female Luer lock fitting.
- **FLF7CO2LUER** is compatible with FUJIFILM 700 endoscopes and insufflators fitted with a female Luer lock fitting.
- **FLOCO2FUJI** is compatible with Olympus endoscopes and insufflators fitted with a special Luer lock fitting.
- **FLPCO2FUJI** is compatible with Pentax endoscopes and insufflators fitted with a special Luer lock fitting.
- **FLFCO2FUJI** is compatible with FUJIFILM 600 and earlier endoscopes and insufflators fitted with a special Luer lock fitting.
- **FLF7CO2FUJI** is compatible with FUJIFILM 700 endoscopes and insufflators fitted with a special Luer lock fitting.

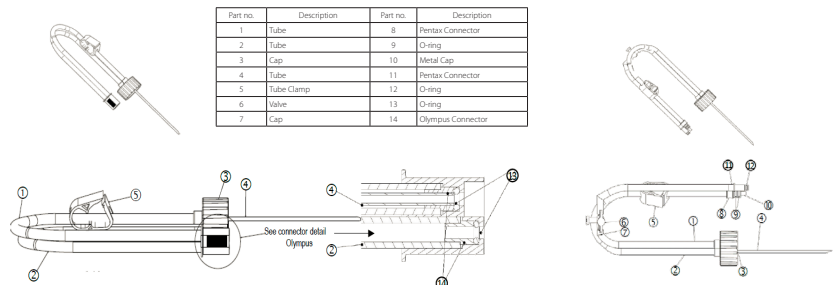CONTRAINDICATIONS:

The contraindications are those particular to any endoscopy procedure.

II. STORAGE / PACKAGING

STORAGE:

- Room temperature: +10°C to +50°C
- Expiry date: 2 years (see on the label)

PACKAGING:

ENDOfube for Wash Bottle CO2 Insufflation, is packaged individually in boxes of 10. Once removed from its packaging, the tubing must be used within 24 hours of opening.

III. WARNINGS AND PRECAUTIONS

The ENDOfube for Wash Bottle + CO2 and Insufflation range:

- Must be used within 24 hours of opening
- Must never be connected or reconnected to an endoscope that has not been washed as per manufacturer instructions.
- Must only be used by qualified endoscopy personnel.
- Not designed to be reused after being used for 24 hours.
- Must be disposed of as waste in compliance with the country's applicable regulations.

Keep an eye on the level of water in the bottle during the procedure. The water bottle must be changed when the water level falls below the tube.

When changing the water bottle, disconnecting the system between two patients, or during any other procedures, authorised personnel must use appropriate techniques to avoid contaminating the ENDOfube for Wash Bottle + CO2 and Insufflation system.

Be sure to identify any leakages in the connector to avoid wetting the floor and causing staff to slip.

The device comes with a **male Luer** lock fitting. In normal conditions of use, the **ENDOfube for Wash Pump** tube is designed to be used away from the patient and close to the endoscopy column.

Special care must be taken to avoid inadvertently connecting the connector to an existing line in the patient with a female connector.

IV. USE / CLEANING / DISINFECTION / STERILISATION

USE:

It is important that you familiarise yourself with the device and complete a trial run before using it in a clinical context (See also the instructions for the endoscope and any accessories used)

1. Open a bottle of sterile water (25cl, 50cl, or 100cl).
2. Open the packaging and remove the **ENDOfube for Wash Bottle CO2 Insufflation** system.
3. Insert the tube into the bottle of sterile water and press the stop to make sure it is watertight.
4. Check that the clamp is in the open position.
5. Connect the air/water connector to the endoscope.
6. Connect the tube with filter of the **ENDOfube for Wash Bottle CO2 Insufflation** to the insufflator.
7. Turn on the processor's light source and then turn on the insufflator.
8. If you are using air, check that the valve on the **ENDOfube for Wash Bottle CO2 Insufflation** tube is open so that air can move from the water bottle to the endoscope.
9. If using CO2, you must turn off the endoscope's air pump and check that the valve on the **ENDOfube for Wash Bottle CO2 Insufflation** tube is open so that the CO2 can move from the insufflator to the endoscope and water bottle.
10. Before starting the procedure, and irrespective of the procedure used (see points 8 and 9), check that it is working properly by starting the endoscope's air/water or air/CO2 canal. If there is a malfunction, check that the water bottle is properly sealed and that the tubes are properly connected to the various devices.
11. If you wish to change the source of insufflation (air or CO2) during the procedure, refer to points 9 and 10.
12. If you need to replace the water bottle, use suitable techniques.


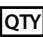
13. Once the procedure is complete, clamp and unplug the **ENDOtube for Wash Bottle CO2 Insufflation** system.
14. You can then turn off the processor and insufflator once you have unplugged the **ENDOtube for Wash Bottle CO2 Insufflation**.
15. Make sure that the device is properly disposed of within 24 hours of opening the packaging. A traceability label is provided for this purpose. Dispose of the device according to applicable regulations in your country.

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CLEANING / DISINFECTION / STERILISATION:

It is a non-sterile device for 24-hour use that cannot be sterilized. Disinfection or cleaning is not necessary.

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