

TENS ELECTRODES



REUSABLE ELECTRODES FOR ELECTRICAL STIMULATION

VERMED[®]
A Nissha Company

What is TENS?

Transcutaneous electrical nerve stimulation (TENS) is a non-invasive, nonpharmacological electrotherapeutic procedure used for pain control. TENS involves the application of electrical current through electrodes placed on the skin through the accompanied use of the BioStim Plus[®] TENS Unit or similar TENS Units.

How does it work?


TENS works by increasing endogenous inhibition and reducing central excitability. More specifically, low-frequency TENS (1-10 Hz) activates μ -opioid receptors, and high-frequency TENS (50-150 Hz) activates δ -opioid receptors.

TENS Placement

TENS electrodes may be used on various places on the body:


- Back of Neck
- Elbows
- Knees
- Foot & Ankle
- Hips
- Lower Back
- Wrists

TENS Placement

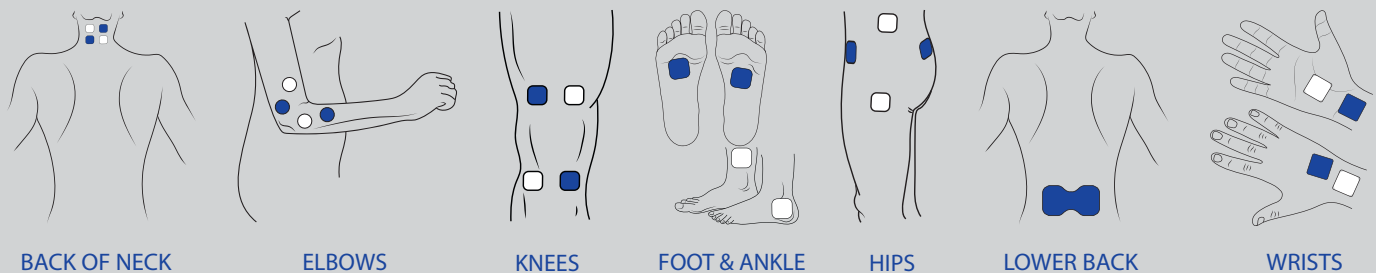

TENS Electrode Anatomy

Patented multi-layer adhesive hydrogel eliminates the performance problems with single-layer hydrogels. This multi-layer approach ensures maximum durability, longevity and performance.

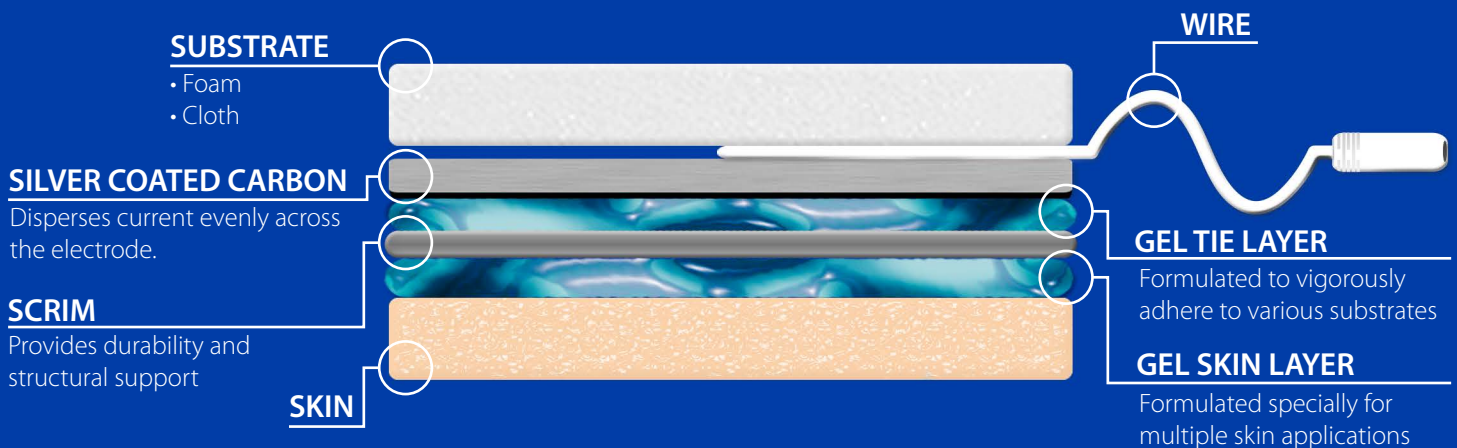


REUSABLE ELECTRODES FOR ELECTRICAL STIMULATION

BioTENS™ - Woven Cloth Electrodes

- Flexible, breathable and comfortable
- Low profile design conforms to body contours for active lifestyles

PART #	SIZE	SHAPE	COLOR	PER POUCH	PER BOX	PER CASE
BF-2	2" x 2"	Square	White	4	80	320
BF-3	1.75" x 3.75"	Rectangle	White	4	80	320
BF-4	1.25"	Round	White	4	80	320
BF-5	2"	Round	White	4	80	320
BF-6	3"	Round	White	4	80	320
BF-7	1.25" x 2.25"	Oval	White	4	80	320
BF-8	2" x 4"	Oval	White	4	80	320
TPWT4-2	2" x 2"	Square	Tan	4	80	320
TPWT4-3	1.75" x 3.75"	Rectangle	Tan	4	80	320
TPWT4-4	1.25"	Round	Tan	4	80	320
TPWT4-5	2"	Round	Tan	4	80	320
TPWT4-6	3"	Round	Tan	4	80	320
TPWT4-7	1.25" x 2.25"	Oval	Tan	4	80	320
TPWT4-8	2" x 4"	Oval	Tan	4	80	320
200-11	6" x 3.25"	Butterfly	Blue	1	20	80

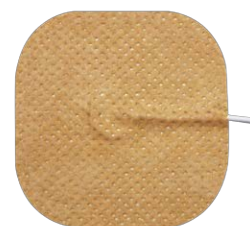

DuraTENS™ - Foam Electrodes

- Durable foam substrate is moisture resistant

PART #	SIZE	SHAPE	COLOR	PER POUCH	PER BOX	PER CASE
SS-2	2" x 2"	Square	White	4	80	320
SS-3	1.75" x 3.75"	Rectangle	White	4	80	320
SS-5	2"	Round	White	4	80	320
SS-6	3"	Round	White	4	80	320



Features

- Easy application and removal
- Reusable
- Various shapes and sizes
- Reusable pouch for easy storage



TENS Unit

This two channel device incorporates touch proof design, digital keypad, large LED display screen, four preset protocols, and is run on two AA batteries.

Features:

- Timer
- Large (2") LCD Display Screen
- Patient Lock/Compliance Meter

Packaging



Resealable Pouch



Global Headquarters

400 Exchange Street
Buffalo, NY 14204
Phone: 1.800.669.6905
Fax: 1.888.658.4941
Vermed.com

3-13-01-009 (10/2016)